



Brike Quick Start Instructions

Series 2 Wheelchair model



Locking brake (for parking, loading and unloading)

brake lever



Red paddle

Black paddle

To engage the lock – push the black paddle down, and then squeeze the brake lever.

To disengage – push the red paddle down, then pull back the black and red paddles together.

Wheelchair loading (reverse for unloading)


1. Swing the rear frame to the side and lock brakes



2. Lower front stabiliser




3. Unlock and swing open handlebars

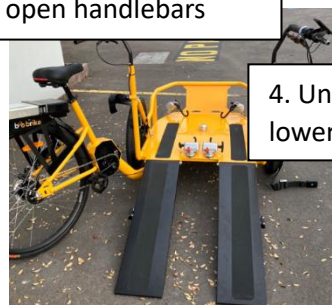

4. Unlatch and lower ramps

5. Load wheelchair and secure with tie down straps

Gears and power assist

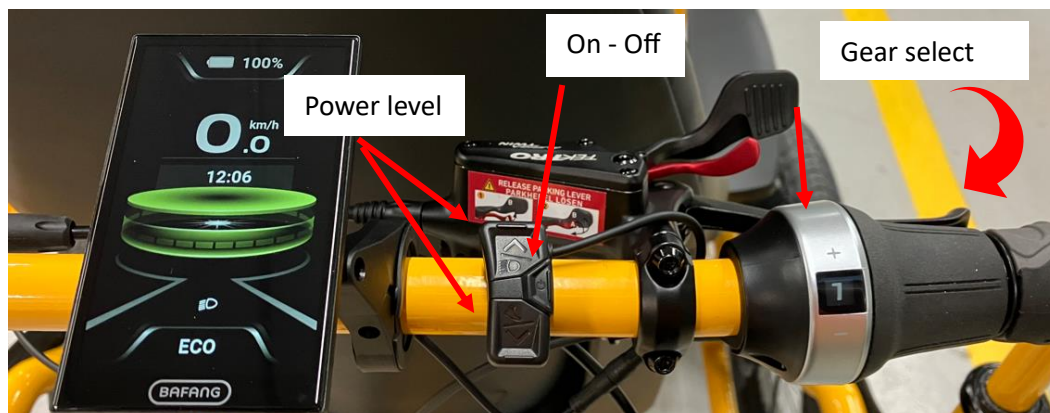

Warning: Tricycles can tip over - Remain seated when riding and slow for corners.

Warning: Keep clear of moving parts - Ensure that there are no loose items hanging that could catch in the wheels or chain.

